# RESMED

## Mirage<sup>™</sup> SoftGel


**NASAL MASK** 

### **ConvertAble**<sup>™</sup> Comfort

The Mirage™ SoftGel Nasal Mask offers exceptional comfort with ResMed's renowned performance and quality.

Easy and quick to fit, Mirage SoftGel is part of ResMed's exclusive ConvertAble™ Series, giving you the choice of two

mask cushions, Mirage Activa™ LT or Mirage SoftGel, on one durable frame. Featuring a soft touch with excellent support, Mirage SoftGel adds the comfort, durability and freedom of choice that only a ConvertAble mask can offer.

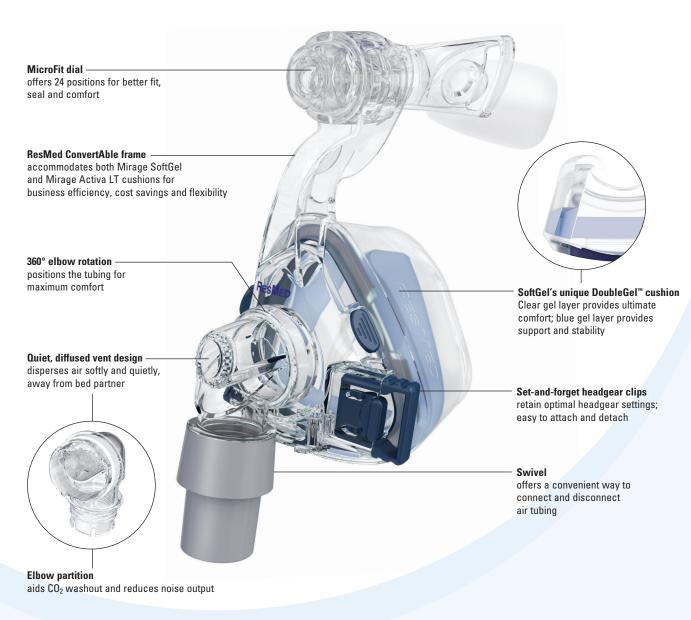



#### **TECHNOLOGY HIGHLIGHT**

#### DoubleGel cushion

Mirage SoftGel's unique DoubleGel cushion creates a soft touch with excellent support.

- **Comfort:** Clear gel layer automatically conforms to the nose and unique facial features of each individual, creating a soft and gentle fit.
- **Support:** The blue gel layer adds stability and support, maintaining its shape and structure.

### **USER BENEFITS**

### **Unparalleled softness and comfort**

- The DoubleGel cushion's outer layer provides unparalleled softness. The blue gel layer maintains stability and support
- Fitting is a simple task—for both current users and new patients

### **Ultra-quiet therapy**

 Special array of small, built-in vents diffuse air quietly. That means a good night's sleep for patients and their bed partners

### **Correct fitting made easier**

- Combination of the Mirage dual-wall cushion with the DoubleGel cushion design makes fitting simple
- The cushion adapts to individual facial contours, creating a comfortable fit with ease

### ConvertAble mask means freedom of choice

- ResMed ConvertAble masks offer the choice of two different cushions on one frame
- Freedom of choice helps patients accept their therapy early and stay engaged in treatment

### **PROFESSIONAL BENEFITS**

### ConvertAble means inventory efficiencies

- SoftGel is a multi-patient, multi-use mask suitable for repeated use in the lab
- Interchangeable and shared components promote inventory efficiencies and cost savings
- Fitting and training time is reduced
- Mirage SoftGel is extremely durable, ensuring consistent performance and fewer callbacks

### Customer satisfaction and added compliance

- Increased therapy acceptance through firsttime fit—long-term comfort
- Interchangeable cushions improve patient compliance while avoiding the cost of expensive mask replacements
- Choice of comfort and fit ensures patient satisfaction—minimizes callbacks
- Comprehensive user support package includes travel bag and user guide with mask fitting and care instructions

### PATIENT-PREFERRED

- 8 out of 10 patients rate SoftGel better in comfort and durability than other gel masks<sup>1</sup>
- 3 out of 4 patients rated their sleep better with SoftGel<sup>1</sup>



## PRODUCT CODES Mirage SoftGel Nasal Mask

Small	61600
Medium	61601
Large	61602
Large Wide	61603

### PRODUCT CODES

### **Reimbursement Codes**

nonnbaroomon oodoo	
Nasal mask use with positive airway pressure device, 1 per 3 months	A7034
Replacement cushion for nasal mask, 2 per month	A7032
Headgear used with positive airway pressure device, 1 per 6 months	A7035

# Mirage<sup>™</sup> SoftGel

**NASAL MASK** 

### **Components Card**

Complete system

S 61600 M 61601 L 61602 LW 61603

Frame system\* (no headgear)

61626 (S) 61627 (M)


61628 (L) 61629 (LW) Frame assembly\*

(no headgear, no cushion)

60196 (S) 60175 (St) **Elbow assembly** 

(elbow, swivel)

16399

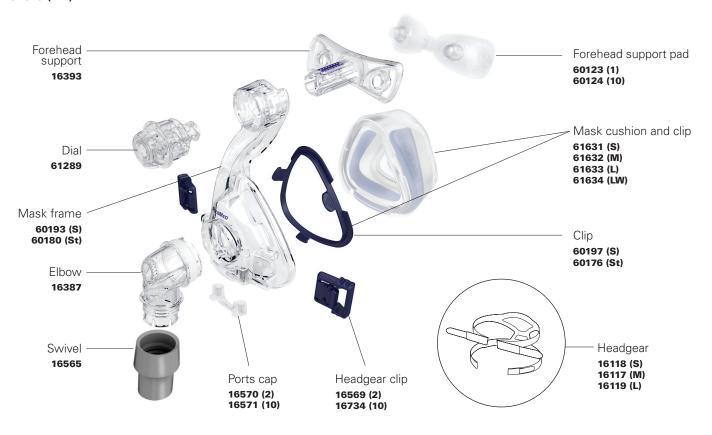

S Small

M Medium

L Large

LW Large Wide

St Standard

\* contains headgear clips



# RESMED

## Mirage<sup>™</sup> SoftGel

**NASAL MASK** 

### **Fitting Guide**



 Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



Adjust the upper straps evenly.
 Do not over-tighten.



• Adjust dial to correct leak at nose.



 Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.



Adjust the lower straps to a comfortable position.



 Adjust bottom straps to correct leak at top lip.



 Reconnect the lower headgear clip, ensuring it clicks into place.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

ResMed Corp San Diego, CA, USA +1 858 836 5000 or 1 800 424 0737 (toll free). ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). See www.resmed.com for other ResMed locations worldwide. ConvertAble, DoubleGel and Mirage Activa are trademarks of ResMed Ltd. Activa and Mirage are trademarks of ResMed Ltd and registered in the U.S. Patent and Trademark Office. ©2009 ResMed Ltd. Specifications may change without notice. 1013468/2 09 10